

Instructions for using the SPIbelt with the myHalo Clip

Caution: When inserting the myHalo belt clip transmitter into the SPIbelt, or removing the SPIbelt from your waist, remember that the myHalo Clip is sensitive to motion. Care should be taken when handling to avoid accidentally triggering the fall detector. Also, take care to avoid unintentionally pressing the square panic button on the front of the transmitter. This may result in an unnecessary fall or panic alert.

Step1: Make sure the battery charger port on the belt clip transmitter is closed.

Step2: Place the myHalo Clip transmitter inside the SPIbelt pouch so the clip side will have the same orientation that it would if you were wearing it on your belt, or waist band, with the panic button facing away from your body. (See Figure 1) **Close the zipper and move it below the face of the transmitter (zipper facing the floor) so it does not rest over the panic button. (See Figure 2)**



Figure 1



Figure 2

Showering with SPIbelt

- Before using the SPIbelt while showering, check to make sure that the SPIbelt zipper is below the face of the myHalo Clip (zipper facing the floor), **to prevent water from entering the pouch through the zipper.**
- After showering, remove the belt clip transmitter from the SPIbelt and **leave the SPIbelt off to dry, pouch inside out. If you have purchased a second SPIbelt, you should use it at this time.**

Two ways to remove Clip from pouch:

- Move the zipper to the top of the SPIbelt (facing up). Unzip and remove the myHalo Clip transmitter before removing belt.
- Or, hold the pouch end of the strap when unclasp to avoid an unnecessary fall alert which may occur if the Clip is dropped rapidly.

Note: To order additional SPI belt's contact Halo at 1-888-971-4256 to order Part Number 31001009-1 (24" to 40" waist) or Part Number 31001009-3 (40" to 55" waist)